



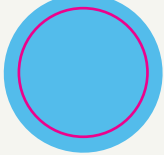


SIZE S: 17mm

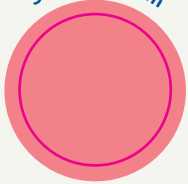

SIZE M: 20 mm



SIZE L: 23 mm

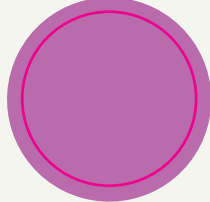

Nipple Shields

Relieve discomfort during breastfeeding

- The innovative butterfly shape offers maximum skin contact between mum and baby
- Features two slits to help mum's milk flow through the shield
- Made of thin, highly flexible SkinSoft™ Silicone (0.3 mm) to make baby comfortable while feeding
- Dimples allow air to flow between shield and baby



SKIN
SOFT™
SILICONE



BPA/BPS
FREE*

*All MAM products are made from materials free of BPS and BPA (BPA Free in accordance with Regulation (EU) No.321/2011.)

If you have any questions, get in touch with our customer service team by email via service4professionals@mambaby.com





Nipple Shields


Find the perfect size for you

Steps to follow:


1. Measure your nipple after breastfeeding or pumping
2. It's the size of the nipple, and not the areola, that's important (Represented by the **green** line on the diagram)
3. Find the right size by using our different sized testers (coloured circles)

TIP: If measurement is not possible after a feed, then add an additional 2 mm to the nipple size.


Nipple shields are a short-term breastfeeding aid.
To be used in consultation with a midwife or health consultant.

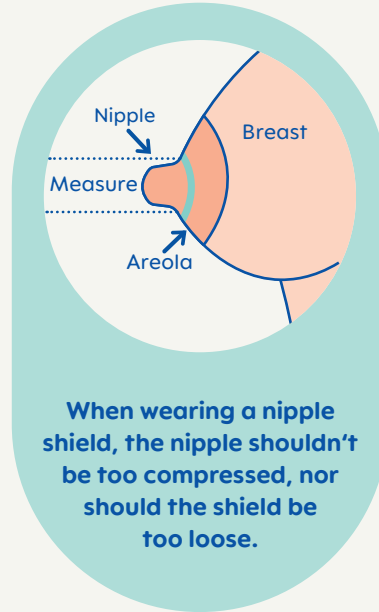

SIZE S: 17 mm

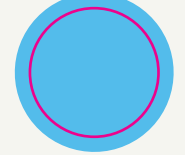

SIZE M: 20 mm

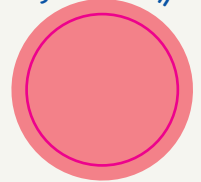

SIZE L: 23 mm

